

## IMPORTANT INFORMATION ABOUT CHARGING THE BATTERY OF MCELL3 DEVICE

MCell3 products are equipped with a Li-ion battery, which you should pay special attention to in order to prevent premature aging and maintain best performance.

According to the battery manufacturer, Li-ion batteries require regular maintenance charging every 3 months. Therefore, we recommend the following measures:

- Remember to charge MCell3 devices regularly, at least every 3 months. For storage purposes optimum charging level is approx. 60-80% out of maximum capacity. You can check the charging level using Muscle Monitor's "Sync MCell" or ErgoLink's "MCell management – function.
- If the next use of the MCell3 device is not known, then after a previous measurement, MCell3 should be charged before storage.
- MCell3 device has a built-in charging protection circuit. However, it is recommended to use USB chargers which are intended to give max. 500mA charging current. Use of faster charging can cause more heat and stress on the battery, which speeds the rate of aging.
- In general, it is not harmful to interrupt charging when the battery is at 80% charge level, especially if you do not need a fully charged battery. This can help reduce the strain on the battery.
- Avoid exposing the battery to extreme temperatures. Li-ion batteries work best at ambient temperatures of around 15-25°C.
- Keep the MCell3's USB terminal clean and free from dust and dirt. This can help ensure a good connection between the battery and charging device.

In case the MCell3 battery is empty and does not start charging, you can try following steps:

- Keep the MCell3 device in slow charging mode (max. 500mA charging current) for several days.
- Press the grey button and disconnect and reconnect the USB cable time to time, which may trigger the charging.
- If those steps do not solve the problem, please send the MCell3 to Myontec for repair.

It's important to remember that while these tips can help extend the life of your Li-ion battery, it will still inevitably degrade over time. The warranty for the rechargeable battery is 6 months. Otherwise, the warranty of MCell3 device is 2 years.

For more information, please contact [support@myontec.com](mailto:support@myontec.com).

- - - - -

## IMPORTANT INFORMATION ABOUT CHARGING THE BATTERY OF MCELL3 DEVICE

MCell3 products are equipped with a Li-ion battery, which you should pay special attention to in order to prevent premature aging and maintain best performance.

According to the battery manufacturer, Li-ion batteries require regular maintenance charging every 3 months. Therefore, we recommend the following measures:

- Remember to charge MCell3 devices regularly, at least every 3 months. For storage purposes optimum charging level is approx. 60-80% out of maximum capacity. You can check the charging level using Muscle Monitor's "Sync MCell" or ErgoLink's "MCell management – function.
- If the next use of the MCell3 device is not known, then after a previous measurement, MCell3 should be charged before storage.
- MCell3 device has a built-in charging protection circuit. However, it is recommended to use USB chargers which are intended to give max. 500mA charging current. Use of faster charging can cause more heat and stress on the battery, which speeds the rate of aging.
- In general, it is not harmful to interrupt charging when the battery is at 80% charge level, especially if you do not need a fully charged battery. This can help reduce the strain on the battery.
- Avoid exposing the battery to extreme temperatures. Li-ion batteries work best at ambient temperatures of around 15-25°C.
- Keep the MCell3's USB terminal clean and free from dust and dirt. This can help ensure a good connection between the battery and charging device.

In case the MCell3 battery is empty and does not start charging, you can try following steps:

- Keep the MCell3 device in slow charging mode (max. 500mA charging current) for several days.
- Press the grey button and disconnect and reconnect the USB cable time to time, which may trigger the charging.
- If those steps do not solve the problem, please send the MCell3 to Myontec for repair.

It's important to remember that while these tips can help extend the life of your Li-ion battery, it will still inevitably degrade over time. The warranty for the rechargeable battery is 6 months. Otherwise, the warranty of MCell3 device is 2 years.

For more information, please contact [support@myontec.com](mailto:support@myontec.com).